

## How to use face shield frame



1. Please prepare a film, rubber band, rings (2 pieces), and stapler.



2. Thread the ring through the hole at the end of the frame. The ring size is about 15 mm.



3. Pass the rubber band from the inside to the outside and put out about 2 cm.



4. Fasten the stapler from inside to outside so that the needle tip does not stick to the head.




5. Insert the film through the slit on the front of the frame. Make the film pop out 1 cm.



6. The stapler is attached to the protruding part to prevent it from falling off.



7. You may attach a cushion to the forehead part.

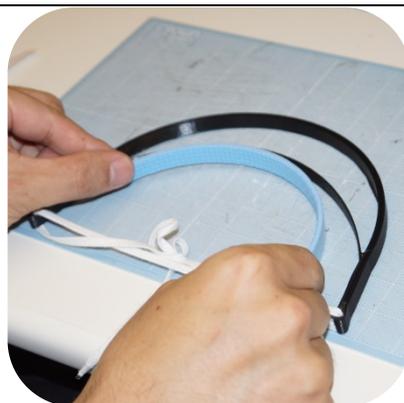

8 A cushion with a width of 8 mm, length of 20 cm is suitable.



9. Attach the completed frame.

\* This frame is donated to medical institutions and medical personnel (free gift).

\* Stationery can be substituted for the film. Since stationery is easy to shake and distorted, it causes seasickness. We recommend using PET films made by chemical manufacturers as much as possible.

\* All parts can be purchased through online shopping such as Amazon.



## フェイスシールドホルダ(フレーム)の使い方



1. フィルム(A4~A3 判程度)、帯ゴム(25cm)、リング(2個)、ステープラーをご用意ください。



2. ホルダ両端にあるホールにリングを通します。リングは13mm程が丁度良いサイズです。



3. リングに織ゴムを内から外へ、頭側が平らになるようにゴムを通し2cm程出します。



4. 織ゴムをステープラーで留めます。針先が頭に当たらぬよう内から外に打ち込みます。




5. ホルダ前部のスリットにフィルムを通して挟みます。フィルムが1cm程飛び出すようにします。



6. フィルムが脱落しないよう、飛び出した1cmの部分にステープラーを打ち込みます。



7. 任意ですが額に当たる部分にクッション材を取付けます。

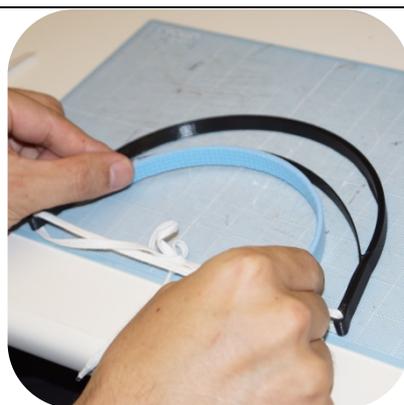

8. 粘着付クッション材を幅8mm程、長さ20cm程で切り、貼ります。



9. 完成したホルダを装着します。

※. 本ホルダは医療機関・医療従事者には無償提供(寄贈)しています。ご希望の場合はNES株式会社(西謙一)

お問い合わせください。3D造形の都合ですぐにご用意できない場合がありますが、ご容赦ください。

※. フィルムは文具を代用できますが酔います。化学メーカー製の硬質透明フィルムをお薦めします。

※. 周辺部材は100均で揃います。写真に出てくる幅25mmの織ゴム、直径13mmの二重リング、子供用ぶつかり防止クッションテープはダイソーで調達しました。

